

Release information

We have released a new version of VisiQuick! The release is ready for download on our FTP-site. The web site is updated with new download links, and we posted an announcement. Note that you will need to have an up-to-date license to install this update. For the most recent updates, visit citodent.com/visiquick/releases.

Current release

Release V4.4.891

Version	Type	Description
V4.4.891 06-09-21	Improved	We fixed a problem in "SetupDatabase.exe", where the Firebird SQL database server configured the incorrect port number 3050.

Release V4.4.890 known issues

Version	Type	Description
26-08-21	Solution	For Firebird SQL installations in version V4.4.890, use this workaround: Remove the "#" in "#RemoteServicePort = 30500" in the file "firebird.conf" in the folder "C:\Program Files\Firebird\Firebird_3_0_vq\", and restart the service.

Release V4.4.890

Version	Type	Description
V4.4.890 24-08-21	New	We have added a License Update Server. The license will now automatically update itself before it expires
	New	Manuals in these languages has been added: Danish, English, French, German, Italian, Netherlands and Norwegian
	New	Support for MyRay CT-Link through VDDS
	New	Support for running multiple instances
	Fixed	Images from all Windows devices can now be imported
	Fixed	Fixed issue with tooth codes
	Fixed	Fixed issue with updating clinic ID in Firebird
	Fixed	Fixed issue with minimize button when VisiQuick is started from the link
	Fixed	Fixed issue with patient property being active when no patient was selected
	Fixed	Fixed issue with manual inserting images when direct replication is active
	Fixed	Fixed issue with PAN status not being set when dragged to PAN
	Fixed	Fixed issue with VQD import showing the wrong patient ID format
	Fixed	Fixed auto replication error when using Paradox

Fixed	Fixed issue with Kodak CT "Use CS Model" setting not being persistent
Fixed	Fixed issue with Carestream open SDK folder button
Fixed	Fixed issue with not reading VQD file when double clicked
Fixed	Fixed issue with back/X was sending VisiQuick to background although this was disabled in the settings
Fixed	Fixed issue with sort order persistence in the patient list
Fixed	Fixed issue with linking and tooth codes
Fixed	Fixed issue with Vista Scan Easy and mirrored last saved images
Fixed	Fixed issue with image import under Windows XP
Fixed	Fixed issue with using multiple tooth codes
Fixed	Fixed issue with .SIP files in Sidexis 4
Fixed	Fixed issue with interactive buttons being reset when change blink mode
Fixed	Fixed issue with image menu items being visible when no image is loaded
Fixed	Fixed issue with PAN/CEPH views and placing a thumb in the color view mini compare view
Fixed	Fixed issue with interactive buttons losing state after opening and closing images
Fixed	Fixed issue with switching to color status after rotating a color image.
Fixed	Fixed issue with sending email and TLS version
Fixed	Fixed issue with registering bszip.ocx
Fixed	Fixed issue with missing files when downgrading from version V4
Fixed	Fixed issue that removed all *.dll files from old installation
Fixed	Fixed issue with secure mode. Secure mode can now be enabled from setup database
Fixed	Fixed issue with default language and tooth settings
Fixed	Fixed issue with telephone and fax column width in Firebird
Fixed	Fixed issue with overwriting the secure mode setting
Fixed	Fixed issue with hidden add, edit and delete buttons
Fixed	Fixed issue with add/remove server buttons
Fixed	Fixed issue with writing to log file
Fixed	Fixed issue with auto start

Previous releases

Release V4.3

Version	Type	Description
V4.3.874 20-12-20	New	Manual and silent installations with MSI update/setup
	New	Support VDDS linking (images stages 1-5 implemented)
	New	Improved support child tooth positions
	New	Device added: Dexis Titanium sensor
	New	Device added: Kavo IXS sensor
	New	Image acquisition now possible for child tooth positions
	New	Support for Azure cloud MSSQL databases
	New	Improved import for KAVO/Palodex NCI interface
	New	Device added: Cruxell Cruxcan
	New	Image view added: 4x3 grid
	New	Integration with 3Shape Dental Desktop
	New	Integration with Boneprox portal
	New	Device added to remote Xray: FireCR
	New	Support for Firebird database
	Fixes	Numerous minor problems solved.