

VisiQuick News August 2024

Release: 4.8.0



Dear VisiQuick user,

Today we have released a new release of VisiQuick, version 4.8.0. This release contains some corrections and additions to the earlier VisiQuick versions. With this letter, we would like to inform you about these corrections.

Contents

Prior to installation	1
Use of replication and VisiQuick 4.7.X and higher.....	1
Corrections in this version	1
Additions in this version	2

Prior to installation

If you work with a VisiQuick version 4.6.x or lower it is of utmost importance with this update that all VisiQuick installations and any replication client are all updated to the new VisiQuick version at the same time. So you cannot work with different VisiQuick versions interchangeably!

This only applies when you are using a VisiQuick version 4.6.x or lower!

Use of replication and VisiQuick 4.7.X and higher

When using Replication and a VisiQuick version 4.6.X or lower, all VisiQuick installations as well as the replication client and server should be updated to this new version.

An older Replication server in combination with a 4.7.X or higher replication Client will give the error message "Function GetDbInfo not implemented" and no longer function.


Unfortunately, this cannot be overcome because the older version is simply not yet aware of the existence of this new function.

The update procedure should be completed in the following order:

- 1) Update all replication server service installations
- 2) Update the replication client installation
- 3) Update all VisiQuick installations using any of the databases to be replicated or received. (In case of multiple locations, i.e. at ALL locations).

If in any doubt, contact your ICT manager, your VisiQuick distributor or the VisiQuick helpdesk!

If you are in doubt whether you are using replication, check this via menu Help - Licence.

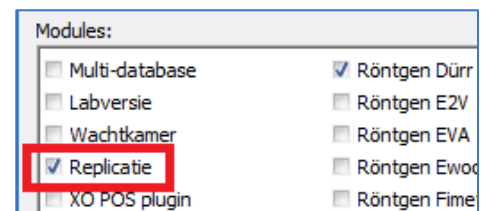

Corrections in this version

- We have fixed an issue with VistaScan in remote X-ray where "always stay on top" checkbox in the configuration screen did nothing.
- We have added a warning message when all images are moved from one patient to another and external images that cannot be moved, may exist.

VisiQuick News August 2024

Release: 4.8.0



- We have fixed an issue with the Palodex CT OnDemand module where some settings weren't saved.

Additions in this version

- We've added functionality that allows you to copy text from the log windows to the clipboard.
- We have added support for the Trident Gold sensor. (Vatech – Trident Iview Gold Sensor)
- We have added support for the CareStream 9600 (2D and 3D acquisition through Trophy Acquisition)